


Avizia CA300 Clinical Telehealth Cart Guide

AVIZIA CA300 CLINICAL TELEHEALTH CART

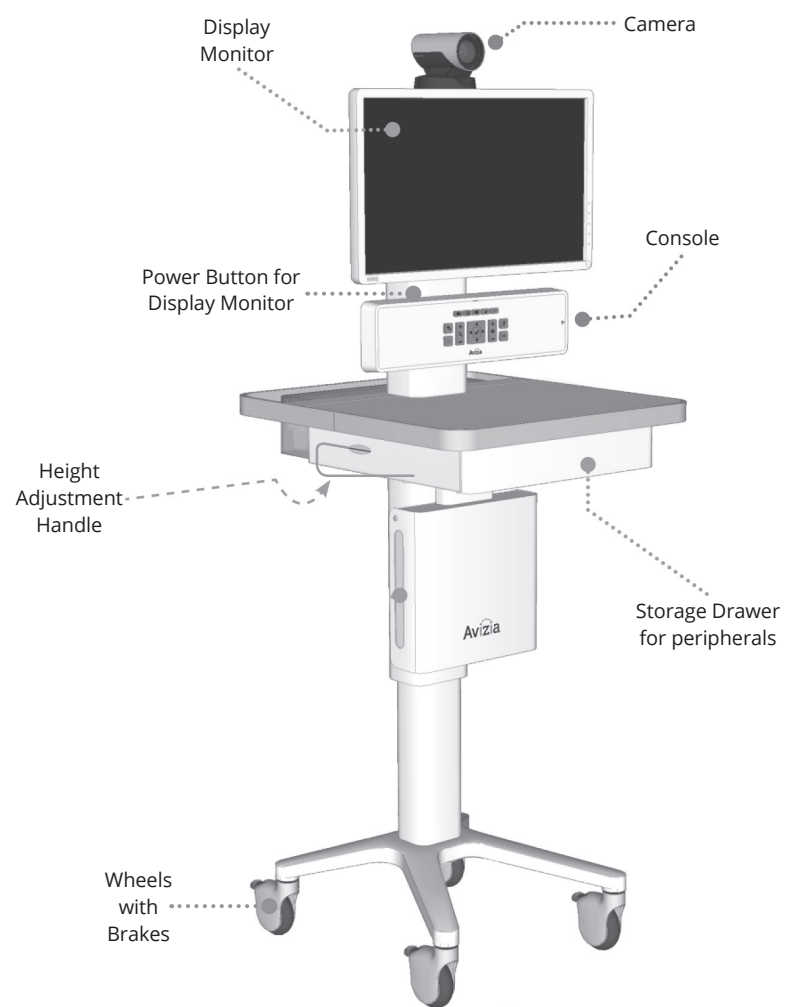


ATTACHMENTS




POWER INDICATOR

- Blue coloured light**
Battery is more than 25% full when the unit is ON or in STANDBY.
- Orange coloured light**
Battery is less than 100% full when the unit is OFF and connected to the charger or computer.
- No light**
Unit is OFF and not connected to the charger or computer.

AVIZIA CA300 CONSOLE

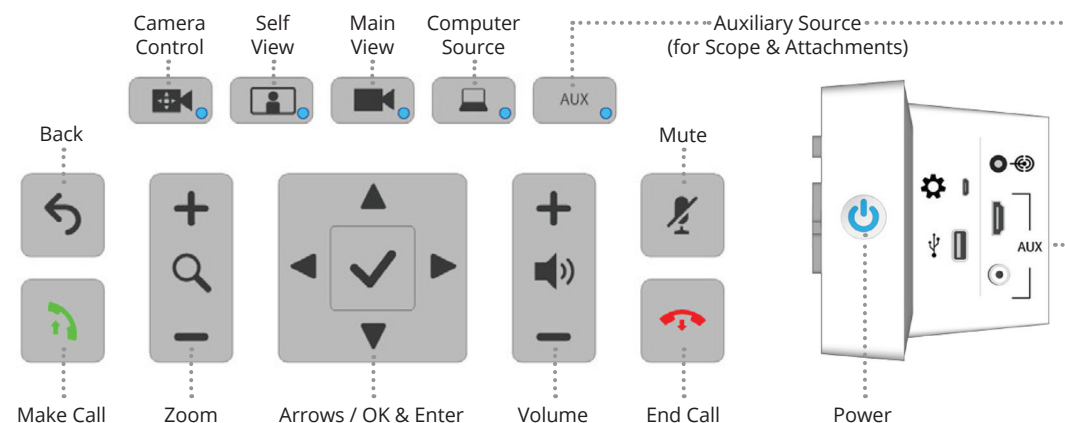


USER INTERFACE

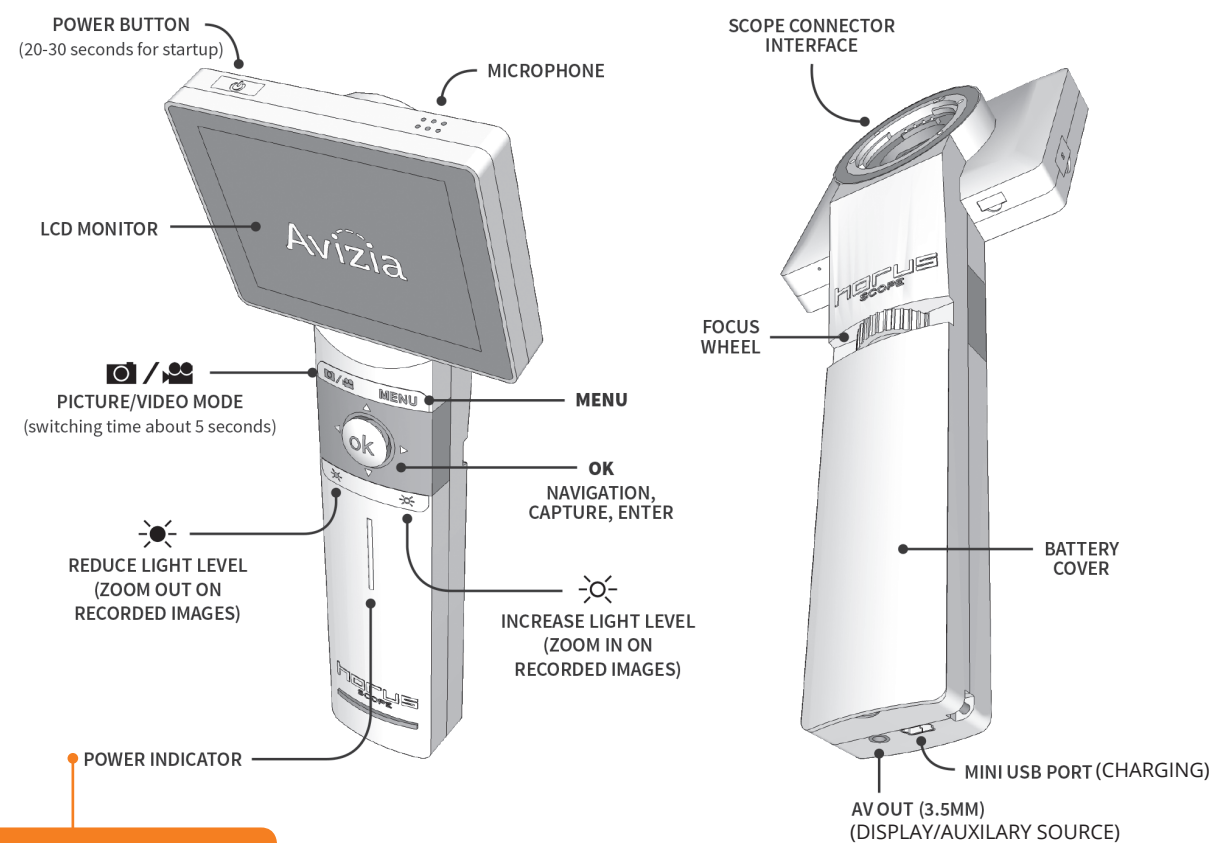

CONNECTING A LENS

1. Press the Power button and allow for 20-30 seconds for the unit to power up.
2. Align the scope assembly alignment guides for both the Optical Scope and the Control Unit.
3. Insert the Optical Scope into the scope connector interface of the Control Unit.
4. Rotate the Optical Scope gently, until it stops, to fasten as shown.
5. Connect the cable to the AUX input and press the AUX button.

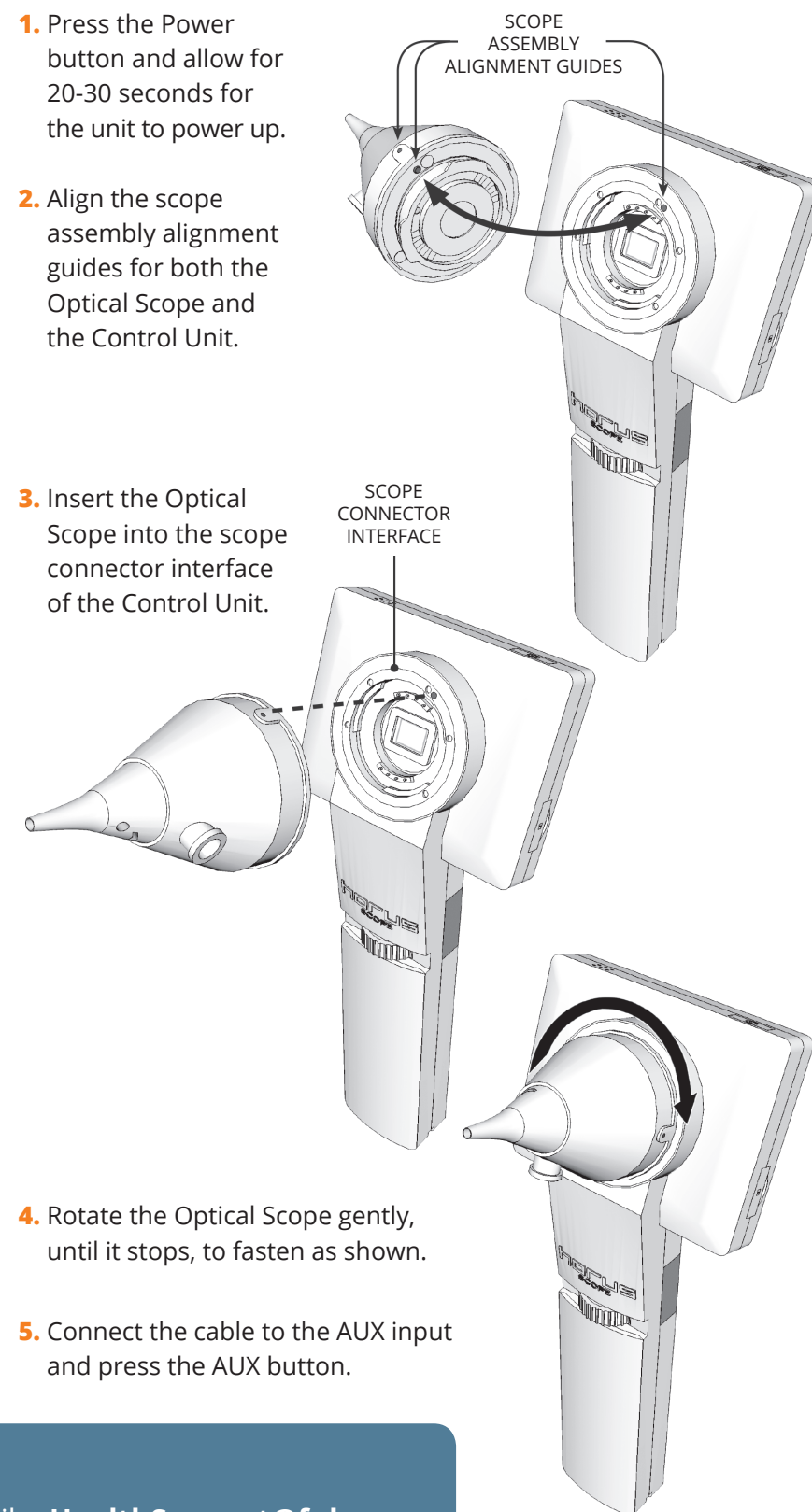

Questions?

Please call the FNHA help desk: 1-844-364-7878 or e-mail: eHealthSupport@fnha.ca

AVIZIA CA300 Clinical Telehealth Cart Guide

HOW TO USE THE DIRECTORY

- Press the **Directory** key on the remote.



- Use the arrow keys to navigate to **My Contacts** and press **Enter**.



- Select the site you wish to call, press the **Call** button.



HOW TO MAKE A CALL ON THE VIDEO CONFERENCE UNIT

- Begin to input the call information you wish to dial. A box will appear as soon as you enter the first number.
- When entering an **IP address**, press the * key in between the numbers. This * will turn into a dot (.) after the third * to complete the **IP address**. (E.g. You want to dial the IP address 204.244.62.30)
- Enter 204*244*62*30 on the remote.
- Once you have entered the call information, press the **Call** button.



- You can also make a call by entering the correct videoconference unit **Extension Number** (e.g. 71999) or **Alias** (e.g. FNHA_UK_Cart@fnha.ca).

HOW TO USE RECENT CALLS

- Close all open menus by pressing



- Select the **Call** option




- Select **Recent Calls**
- Use the **ARROW** keys to select the site you would like to call and press the **CALL** button.

CAMERA CONTROLS

- To move your camera around, use the **Arrow** keys to move your camera from side to side or up and down.
- You can zoom in and out with the **Zoom** key located to the right of the arrow keys.

HOW TO MUTE YOUR MICROPHONE

- To mute your microphone you can either press the button on the microphone itself or press the mute button on the remote.
- When the microphone has a **RED** button it has been muted. When the button is **GREEN** it is unmuted and on.

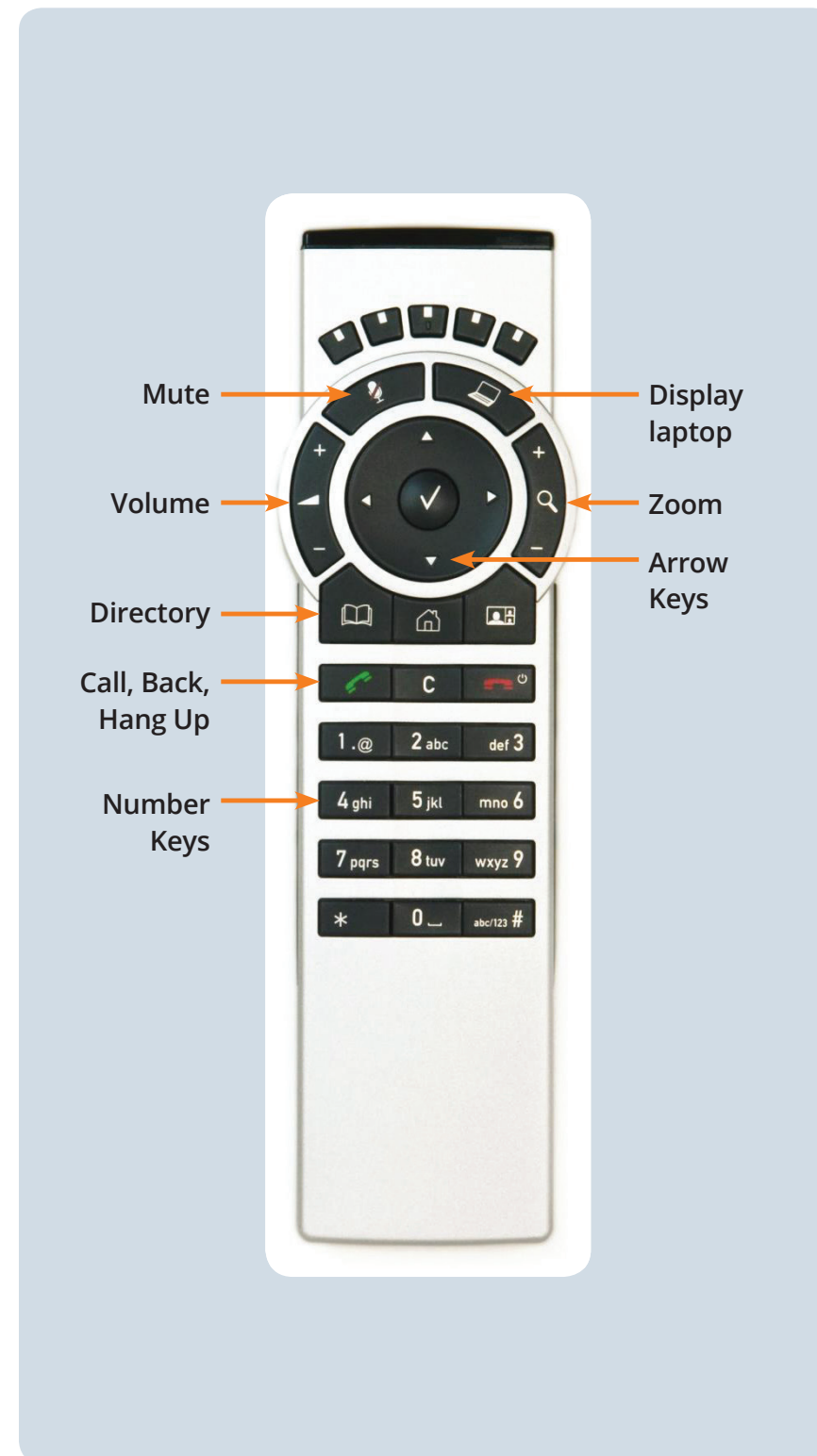

Questions? If you have any questions, please call the FNHA help desk: **1-844-364-7878** or e-mail: **eHealthSupport@fnha.ca**